MASCARILLA AUTOFILTRANTE FFP2

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Organismo Notificado: CCQS Certification Services Limited, Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin 15, D15 AKK1, Irlanda (ON 2834).

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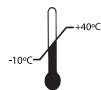
Importer's model: 6753 FFP2 Auto filtering mask Tensil Color  **2834**

Manufacturer's model: ZN9501 FFP2 NR

Non-reusable FFP2 filtering mask according to EN 149:2001+A1:2009.

PERSONAL PROTECTIVE EQUIPMENT.

This item has been manufactured following the requirements of Regulation (EU) 2016/425 for basic use.

Failure to follow the instructions and warnings on the use of this mask during all times of exposure can reduce the effectiveness of the mask and could result in illness or disability.

As with any respiratory device, the wearer must be adequately trained prior to use. Before use, the wearer should always check the mask is in good condition- no dirt, no damage to headharness attachment, etc.

Limitation:

Do not use the respirator or enter or stay in a contaminated area under the following circumstances:

- Atmosphere contains less than 19.5% oxygen.
- If you smell or taste contaminant.
- For protection against gases or vapors.
- Contaminants or their concentrations are unknown or immediately dangerous to life or health.
- For sandblasting, paint-spray operations and asbestos treatment.
- In explosive atmospheres.

Warnings:

- This mask marked "NR", shall not be used for more than one shift.
- Never substitute, modify, add or omit parts in the configuration as specified by the manufacturer.
- This mask helps protect against certain particle contaminants but does not completely eliminate exposure to the risk of contracting disease or infection.
- It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.
- Change the mask immediately if breathing becomes difficult or the mask becomes damaged or distorted.
- Change the mask if a proper face seal cannot be achieved.
- Change the mask if it is removed while in the contaminated areas.

Fitting instructions:

1. Hold the particle filtering half mask in position over the nose and mouth.
2. Pull the headharness behind to ears, attach the headharness to the retaining clip, improve comfort and prevent leakage.
3. Ensure the nose clip is securely moulded around the nose, resting the ends against the cheek to obtain a good seal.
4. To check a proper fit, cup both hands over the mask and exhale vigorously. If air leaks around the nose, tighten the nose clip, if air leaks around the edge, reposition the headharness for better fit.
5. Repeat adjustments until the mask is sealed properly.
6. If a proper seal cannot be achieved, do not enter the contaminated area or it may cause illness.

Storage:

The mask should not be removed from its package unless it is required and should be discarded after use.

Notified Body: CCQS Certification Services Limited, Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin 15, D15 AKK1, Ireland (NB 2834).

Declaration of conformity: It is available at: www.makito.eu (search for item number).

Manufactured by: Anhui Zhongnan Air Defence Works Protective Co., Ltd. Qinshan Comprehensive Economic Development Zone, Anhui Province, China.

Trade mark: Zhongnan

Batch: V0110

Imported by: Mkto Catal Importaciones S.L. Ctra. Huércal-Overa S/N 04640 Pulpi, Almería, Spain

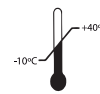
Made in: China

See information supplied by the manufacturer/importer.

2023/10
End of shelf life

Temperature range of storage conditions

Maximum relative humidity of storage conditions < 80%

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Notifizierte Stelle: CCQS Certification Services Limited, Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin15, D15 AKK1, Irland (ON 2834).

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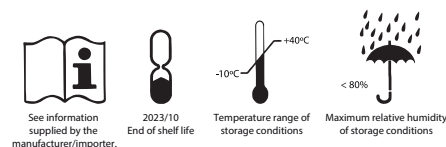
Notified Body: CCQS Certification Services Limited, Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin15, D15 AKK1, Ireland (NB 2834).

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Manufactured by: Anhui Zhongnan Air Defence Works Protective Co., Ltd. Qinshan Comprehensive Economic Development Zone, Anhui Province, China.
 Trade mark: Zhongnan
 Batch: V0110

Imported by: Mtko Catal Importaciones S.L. Ctra. Huércal-Overa S/N 04640 Pulpi, Almería, Spain

Made in: China